

ZOLL®

Cardiac Diagnostics



PATIENT INFORMATION

Mobile Cardiac Telemetry
Cardiac Event Monitoring

ZOLL Cardiac Diagnostics

Patient Information

Why Do I Need to Wear a ZOLL Cardiac Monitor?

Your doctor has prescribed a cardiac monitor for you to understand more about your heart. Abnormal heartbeats, and other symptoms, may not occur regularly so they are not always present while you are at an office visit. In other cases, the abnormal heartbeats may not produce any symptoms, but their detection and treatment are important to your overall health. A cardiac monitor is worn continuously for up to 30 days, while you do your normal activities. The monitor will record your heart rate and rhythm, and provide that information to your doctor, which may help diagnose and treat your condition.

How Does It Work?

The ZOLL Cardiac Monitor is a portable, battery-operated device that continuously records your heart's rhythm and automatically sends the information to your doctor. Once you begin wearing the monitor, it records, detects, and transmits your data without any intervention from you. If you begin to feel symptoms related to why your monitor was prescribed, you can indicate these events by firmly pressing and holding the button on the side of the monitor. The up and down arrows on the monitor will allow you to describe your symptoms and activity level. (Detailed instructions will be included in the User Manual.)



When Do I Start Wear?

After your doctor prescribes the ZOLL Cardiac Monitor, you will receive a package in the mail containing the monitor and associated components. Upon receipt, you should attach the monitor and electrodes to your body to begin monitoring. If you would like assistance with setting up your monitor, please call ZOLL Cardiac Diagnostics toll-free, 24 hours a day, 7 days a week at 1.888.592.3798.

How Do I Start Wear?

Detailed instructions will be shipped with the ZOLL Cardiac Monitor. Follow the Quick Start Guide for easy setup, and refer to the User Manual for more information.

Questions?

If you have any questions about setting up the device, please call ZOLL Cardiac Diagnostics toll-free, 24 hours a day, 7 days a week at 1.888.592.3798.

Within 24 to 48 hours of receiving the shipment, you will receive a call from ZOLL to answer any questions you may have and to ensure proper setup of the ZOLL Cardiac Monitor.

Please note that our caller ID will show the number 1.412.968.3658 from Pennsylvania.

Contact **ZOLL**
24 hours a day, 7 days a week



For ZOLL Cardiac Diagnostics support, please call **888.592.3798**



For more information on ZOLL Cardiac Diagnostics, please email **CardiacDiagnostics@zoll.com**

ZOLL[®]

© 2017 ZOLL Medical Corporation. ZOLL is a registered trademarks of ZOLL Medical Corporation in the United States and/or other countries.

ZCDM-0011